
Getinge HS33 Hospital Sterilizers

GETINGE GROUP

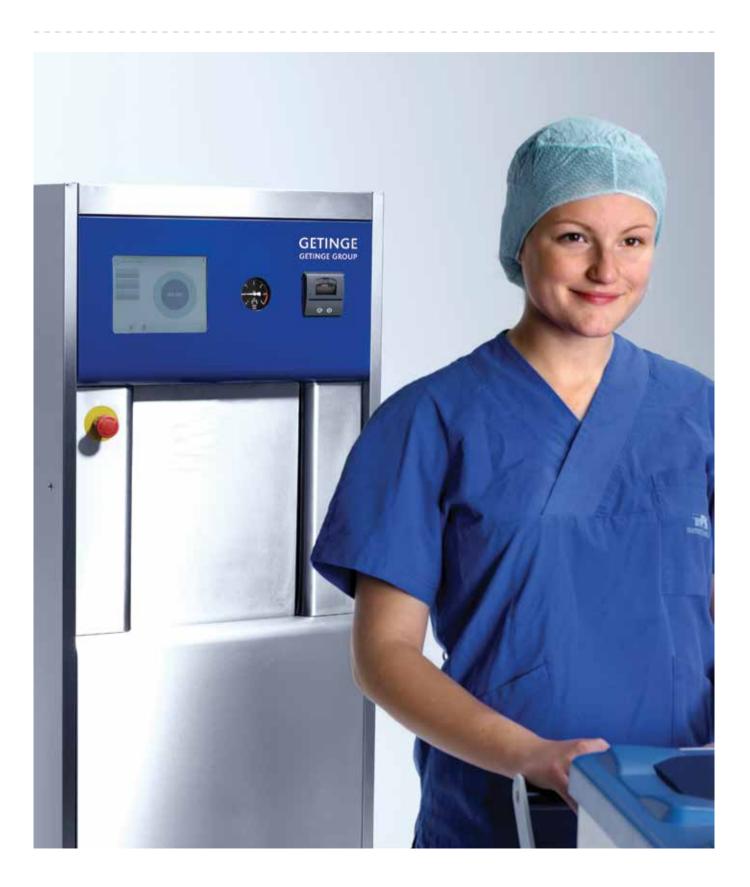

Lights on for the **new generation** of sterilizers

The new Getinge HS33 sterilizer is one of the new breed of state-of-the art sterilizers from the world's leading brand in infection control. Sterilizers that further perfect the efficient performance and superior throughput you can always expect from Getinge. You recognize them by their clear, light and characteristic touch-screen panels, easily readable from a distance. As well as by their thought-through and extremely user-friendly design, making them easier to operate than ever.



Ergonomic and user-friendly

The clear and intuitive interface of the touch-screen panels is only one of many examples of how we ensure that Getinge sterilizers are easier to operate and more ergonomic. Read more about touch screens on page 4.



Very fast processes – superior efficiency

Getinge HS33 has the fastest sterilization cycle on the market and a high loading capacity, which ensures superior production efficiency.

A preset standard process with four pre-vacuum pulses, 4-minute sterilization and post-vacuum for drying takes less than 30 minutes in total.



High-speed Sterilization[™] superior capacity and efficiency

The secret behind the high performance of the Getinge sterilizer is its unique construction, which gives very short and effective cycles.

Getinge color touch screen

Getinge HS33 is equipped with the latest touch-screen technology and an 8.5" colour touch screen for optimal, user friendly control. The touch screen also gives you a user friendly, intuitive interface.

Unique cost-saving construction

- The elements have practically unlimited lifetime, since they are never in contact with water.
- The total power supply is less than 6 kW, of which the steam generator is only 3.6 kW. In most cases, Getinge HS33 can be connected directly to the existing electrical installation.
- The unique construction allows very short cycle times.
- Very few components reduces service and maintenance to an absolute minimum.
- No pressurized steam generator during stand-by.

Eco-system

As a standard feature, the sterilizer is equipped with a watersaving system to reduce water consumption. Your total consumption of water for a standard process is only about 60 liters.

Optional unique water-saving module (VWW)

Additionally, the sterilizer can be equipped with a unique water saver module, which saves between 80% and 95% of the water consumption.



Estimated consumption per process will be between 3 and 12 liters, depending on sterilizer load. By doing so you'll not only contribute to a better environment, you will also lower your running costs.

VWW - Vacuum Without Water

Air compressor

If you have no supply of compressed air, we offer an optional integrated silent and oil-free compressor.





Two different chamber sizes

Getinge HS33 is available in two versions: HS33 with a 1 STU chamber capacity (chamber size 320 x 620 x 320 mm), and HS33+ with a 1.5 STU chamber capacity (chamber size 320 x 971 x 320 mm).

Minimized service

The sterilizer is designed with few components and equipped with Getinge software, making service and maintenance very simple. Connections for external independent temperature and pressure sensors for validation, as well as connections for PC, are part of the system.

Easy to operate

Getinge HS33 is very simple to operate. Close the door and press the start button. A number of accessories ensure simple handling and a good working environment. Getinge HS33 is available in single- and double-door versions, with manual or automatic door opening/closing operation. (The double-door version is only automatic.)

Process documentation

Getinge HS33 is equipped with a built-in printer with independent pressure and temperature sensors. The sterilizer can also be equipped with a USB storage solution. Getinge HS33 can also be connected to T-DOC*.



*T-DOC, Getinge's traceability and handling system that integrates both washer-disinfectors and sterilizers.

Accessories

We offer a wide range of accessories to choose from (see examples below). Further information is available at www.getinge.com



Loading trolley, facilitating the movement of the load into/out of the sterilizer.



Stainless steel rack with rollers for 18 dental cassettes. Note, the rack gives a capacity up to 36 trays per hour.*

Perforated tray, stainless

sizes for HS33 and HS33+

(603/958 x 297 x 15 mm).

steel. Available in two



Rack with rollers with three levels, stainless steel. Available in two sizes for HS33 and HS33+.*



Fixed aluminum rack with three levels. Available in two sizes for HS33 and HS33+.



Basket preferably used together with the tray with rollers. Available in three sizes for HS33 and HS33+.

* For optimized ergonomics, use together with the loading trolley.



Perforated tray with rollers, stainless steel. Available in two sizes for HS33 and HS33+ (600/960 x 310 x 35 mm).*



Perforated tray, aluminum. Available in two sizes for HS33 and HS33+ (583/958 x 293 x 15 mm).


Optional loading system



Loading of modular wire baskets, using the perforated tray with rollers. For one wire basket $590 \times 300 \times 90$ mm, 2-3 baskets $590 \times 300 \times$ 100 mm, or a combination.



Set of shelves used for loading of containers (STU) and individual packs directly on the shelves.

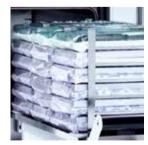

This dental rack enables sterilization of a single load of 18 dental trays, 290 x 190 x 35 mm.



Containers on tray in fixed aluminum rack.

Technical data

Connection of utilities / demand

water	
Quality required	Particle-free, Max 4 dH
Pressure required	Min 3 bar
Consumption	50-60 liters/process
Consumption VWW	3-12 liters/process
Connection	1/2"

Central deionized water or osmosis water

Quality required	Max 30 µS/cm
Consumption	0.5-3 liters/process
An osmosis system can be integrated in the sterilizer	

Compressed air (automatic doors only)

The compressed air module can be integrated in the sterilizerQuality requiredDry, filtrated to 5 micronsPressure4–5 bar

Drain	
Connection	1 1/4"
Max drain	10 L/min

Max 6 kW

Electrical

Consumption

Material of construction

Casing	Stainless steel 304
Pressure vessel	Stainless steel 316 Ti

Standard programs HS33 ~30 min Instruments/Textiles 134 °C ~36 min HS33+ HS33 ~14 min Flash 134 °C HS33+ ~18 min HS33 ~35 min Instruments/Textiles 121 °C HS33+ ~42 min HS33 ~39 min 134 °C 18 minutes HS33+ ~47 min

Test programs: - Bowie & Dick, Leakage test

Process times are load-dependent and approximate. They refer to a complete process including drying with an average load. For more details see Product Specification.

Pressure vessel (single- or double-door version)

Dimensions HS33	620 x 320 x 320 mm
Dimensions HS33+	971 x 320 x 320 mm
Volume HS33	63 L (1 STU)
Volume HS33+	100 L (1.5 STU)
Load	

Ludu	
Instruments, max	15 kg
Textiles, max	7.5 kg

Approvals / Standards

MDD 93/42 ECC, PED 97/23		
Dimensions HS33	595 x 1575 x 880 mm	
Dimensions HS33+	595 x 1575 x 1250 mm	
For exact measurements, see installation drawing.		



GETINGE GROUP

Getinge Infection Control AB P O Box 69, SE-305 05 Getinge, Sweden Phone: +46 10 335 00 00 info@getinge.com

www.getinge.com

MAQUET GETINGE GROUP

ARJOHUNTLEIGH GETINGE GROUP

GETINGE GROUP

Getinge Group is a leading global provider of innovative solutions for operating rooms, intensive-care units, hospital wards, sterilization departments, elderly care and for life science companies and institutions. With a genuine passion for life we build quality and safety into every system. Our unique value proposition mirrors the continuum of care, enhancing efficiency throughout the clinical pathway. Based on our first-hand experience and close partnerships, we are able to exceed expectations from customers – improving the every-day life for people, today and tomorrow.